



**LOOK YOUNGER.
FEEL STRONGER.
LIVE BETTER.**

Scan the QR code to head over to our website, where you can easily reorder, view instructions online or contact us.



CONTACT US

If you have any questions about using your NAD+ AT HOME pen, please feel free to reach out.

We are happy to assist with any queries you may have. You can contact us using the details below:

Address



**55-57 SEABANK ROAD,
WALLASEY, CH45 7PA**

Phone Number



+44 20 3824 2840

Email Address



HELP@NADAID.CO.UK

NAD+
AT HOME

NAD+ PEN

**ENSURE YOU OPEN
THESE INSTRUCTIONS
FULLY & READ
CAREFULLY**

INSTRUCTION GUIDE



NAD+ PEN USAGE

- 1 Gently pull the top section of your NAD+ pen to remove the lid, exposing the clear cartridge holder.



- 2 Twist & unscrew the clear cartridge holder from the pen.



- 3 If the plunger is exposed, gently press it into the base of the pen using your finger.



- 4 Open the cartridge box and take out a cartridge. Insert it into the clear holder, making sure the silver tip goes in first.



- 5 Firmly push and twist the clear cartridge holder into the pen base until secure.



- 6 Take a needle tip from the pack and remove the seal from the bottom of the needle. Carefully but firmly screw it onto the end of the pen. **Do not remove the needle cover just yet.**



- 7 Clean your chosen injection site with an antibacterial wipe. Choose a fatty area such as the stomach, buttock, or thigh for best results.



- 8 You can now remove the plastic needle cover and the green needle safety protector.



- 9 **For first-time use of a new cartridge:** Twist the base of the pen until the dose meter is fully extended.

Next, push the dose meter and watch the plunger rise inside the cartridge holder. Repeat this action until the plunger reaches the grey bung at the bottom of the cartridge, and you feel resistance.

Holding the pen upright, twist the black tip out and push until all air bubbles are expelled, a small drop of liquid will appear at the tip of the needle. Your pen is now ready to use.



- 10 While keeping the pen in hand, twist the base until the pointer aligns with your dosage, typically 30 units (0.3ml/30 mark).



- 11 Gently pinch the skin around the injection site to create a firm but comfortable area for the injection.



- 12 Align the pen with the injection site and press the needle into your skin to ensure proper penetration.



- 13 Firmly press down at the base of the pen to depress the plunger and administer the NAD+. **Keep the pen steady to prevent bruising.** A slight stinging sensation is normal.



- 14 Once the dial returns to zero (0) and the full dose has been administered, hold the pen in place for ten seconds before carefully removing the needle from your skin.



- 15 Carefully place the green needle tip protector back onto the needle.



- 16 With the green needle protector attached unscrew the needle tip from the pen whilst securely holding the cartridge.



- 17 Dispose of the used needle in your sharps bin. Replace the NAD+ pen lid and place the pen and any extra cartridges in the fridge.



DOSAGE

- You will be administering a solution of NAD+ via subcutaneous injection (under your skin).
- NAD+ suggested protocol: 1x 0.3ml subcutaneous injection every day for the first three days. Followed by 1x 0.3ml subcutaneous injection every ALTERNATE day.
- The NAD+ solution has a strength of 500mg per 3ml liquid. The typical advised daily dose is 50mg (0.3ml) which is 30 units on the pen graduations.

STORAGE

- Pens and cartridges should be stored in a refrigerator between 2-8 degrees Celsius (35.6-46.4 Fahrenheit).
- Make sure you use your NAD+ cartridges within their expiry date.